

R-C3 Treatment Program

mode	program	treatment time (min)	Pulse rate (Hz)	Pulse width (us)	Remark
TENS	P01	30	100	150	Gate control
	P02	30	60	200	Gate control
	P03	30	15	260	Low frequency
	P04	30	2-60	260-160	Modulation
	P05	30	60/50/45/10/50/35	200	Modulation
	P06	30	50	180	Gate control
	P07	30	40/6/50	200	Modulation
	P08	30	60	156-260	Modulation
	P09	30	60	260	Modulation
EMS	P10	30	4	200	Active Recovery
	P11	30	5	300	Active Recovery
	P12	30	20	200	Build Endurance
	P13	30	50	200	Muscle Strengthening
	P14	30	60	200	Muscle Strengthening
	P15	30	50	200	Muscle Strengthening
	P16	30	60	200	Muscle Strengthening
	P17	30	70	200	Muscle Strengthening
MASSAGE	P18	30	8	300	Con.
	P19	30	100	220	Con.
	P20	30	28~45	120~250	knead
	P21	30	25~80	120~250	Rub
	P22	30	50~100	100~240	Tap